

Introduction:

Antibodies against different structures of the testicle are reported in male sterility with azoospermia. Anti-Leydig cell antibodies are observed in steroid defects. Antibodies staining the interstitial cells of the testis have been detected in patients with Addison's disease.

Principles:

The primary test reaction involves circulating anti-testicular antibodies present in the patient's serum, which bind to the substrate to form an antigen/antibody reaction. This occurs during an incubation period while the serum covers the antigen substrate surface. The secondary reaction utilizes a fluorescein labelled anti-human globulin conjugate that labels the primary antigen/antibody reaction and can be visualized through the microscope.

Materials Provided:

Storage & Stability of Components:

1. FITC Conjugate No. 1502L (3.0 ml) with Evans Blue Counterstain is to be stored at 2-8°C upon receipt. The conjugate is stable at this temperature until expiration date on the vial label.
2. The antigen slides of primate testes sections must be stored at 2-8°C or lower upon receipt. Check label for specific expiration date.
3. Anti-testes (lyophilized) positive control No. 6802S (0.5 ml) should be stored at 2-8°C upon receipt. The control contains 0.095% sodium azide as preservative. In the lyophilized state, the control sera should be stored at 2-8°C. Refer to the product label for expiration date. Once reconstituted with 0.5 ml distilled water, small aliquots may be stored for 6 months at -20°C or lower or they may be stored at 2-8°C for up to 14 days.
4. Universal negative control No. 1000L (1.0 ml) should be stored at 2-8°C. Check label for specific expiration date.
5. Buffer Pack No. 1601 - Phosphate Buffered Saline is stable at room temperature storage to the expiration date indicated on the label. The

reconstituted Buffer does not contain preservatives and should be stored at 2-8°C. Care should be taken to avoid contamination.

6. Mounting Medium No. 1610 is stable when stored at 2-8°C. Check label for specific expiration date.

Note: All kit components are available separately. Please see the current SCIMEDX Corporation Catalog for more details.

Additional Materials Required but not Provided:

Test tubes and rack or microtiter system
Disposable pipettes
Staining Dish and Slide Forceps
Moisture Chamber
Volumetric Flask (500 ml)
Distilled H₂O
Fluorescence Microscope
Paper Towels - lint free

Reagent Preparation:

1. Buffer Pack No. 1601. Rehydrate buffer with 1 liter of sterile distilled water.

Specimen Collection:

Serological specimens should be collected under aseptic conditions. Hemolysis is avoided through prompt separation of the serum from the clot. Serum should be stored at 2-8°C if it is to be analyzed within a few days. Serum may be held for 3 to 6 months by storage at -20°C or lower. Lipemic and strongly hemolytic serum should be avoided. When specimens are shipped at ambient temperatures, addition of a preservative such as 0.095% sodium azide is strongly recommended.

Test Instruction:

Screening: dilute test serums 1:4 in PBS. **Titration:** set up doubling dilutions of serum starting at: 1:4, 1:8, 1:16, 1:32, etc.

1. Once slides reach room temperature tear slide envelope at notch. Carefully remove the slide and avoid touching the antigen areas. The slide is now ready to use.

2. Place a drop of diluted serum (20 to 30 µl) and controls over the antigen wells.
3. Place slide with patient's serum and controls in a moist chamber for 30 minutes at room temperature (approximately 19-24°C).
4. Remove slide from moisture chamber and tap the slide on its side to allow the serum to run off onto a piece of paper towel. Using a wash bottle, gently rinse remaining sera from slide being careful not to aim the rinse stream directly on to the well.
5. Wash in PBS for five minutes. Repeat using fresh PBS.
6. Place a blotter on the lab table with absorbent side up. Remove slide from PBS and invert so that tissue side faces absorbent side of blotter. Line up wells to blotter holes. Place slide on top of blotter. **Do not allow tissue to dry.** Wipe back of slide with dry lint free paper towel. Apply sufficient pressure to slide while wiping to absorb buffer.
7. Deliver 1 drop (20-30 µl) of conjugate per antigen well. Repeat steps 3-6.
8. Place 4-5 drops of mounting medium on slide.
9. Apply a 22 x 70 mm coverslip. Examine the slide under a fluorescent microscope. Note: To maintain fluorescence, store mounted slide in a moisture chamber placed in a dark refrigerator.

Quality Control:

1. Positive and negative serum controls must be included in each day's testing to confirm reproducibility, sensitivity and specificity of the test procedure.
2. The negative serum control should result in little (+) or no fluorescence. If this control shows bright fluorescence, either the control, antigen, conjugate or technique may be at fault.
3. The positive serum control should result in bright 3+ to 4+ fluorescence. If this control shows little or no fluorescence, either the control, antigen, conjugate or technique may be at fault.

4. In addition to positive and negative serum controls, a PBS control should be run to establish that the conjugate is free from nonspecific staining of the antigen substrate. If the antigen shows bright fluorescence in the PBS control repeat using fresh conjugate. If the antigen still fluoresces, either the conjugate or antigen may be at fault.

Results:

The slide should be examined under 400X, high dry objective. Positive result is observed as a 1 + or greater fluorescent reaction

Limitations of Procedure:

1. No diagnosis should be based upon a single test result, since various host factors must be taken into consideration.

Precautions:

1. All human components have been tested for (HbsAg) and HTLVIII/ LAV by an FDA approved method and found to be negative. (Not repeatedly reactive). However, this does not assure the absence of HbsAg or HTLVIII/LAV. All human components should be handled with appropriate care.
2. The sodium azide (0.095%) included in the controls and conjugate is toxic if ingested.
3. Do not use components beyond their expiration date.
4. Follow the procedural instructions exactly as they appear in this insert to insure valid results.
5. For in vitro diagnostics use.
6. Handle slides by the edges since direct pressure on the antigen wells may damage the antigen.
7. Once the procedure has started do not allow the antigen in the wells to dry out. This may result in false negative test results, or unnecessary artifacts.

BIBLIOGRAPHY:

1. Barbarino-Monnier, P., Bene, M., Gobert, B., Landes, P., Guillet-Rosso, F., Faure, G.: Anti-ovary antibodies, repeated attempts, and outcome of in-vitro fertilization. *Fertil Steril* 56(5): 928-32, 1991.
2. De Castro, M., Jeyendran, R., Zaneveld, L.: Hypo-osmotic swelling test: Analysis of pre-vasectomy ejaculates. *Arch Androl* 2:11-6, 1990.
3. Jeyendran, R., van der Ven, H., Kennedy, W., et al: Acro-someless sperm. A cause of primary male infertility. *Androl* 17(1):31-6, 1984.
4. Moncayo-Neveda, H., Moncayo, R., Benz, R., Wolf, A., Lauritzen, C.: Organ-specific antibodies against ovary in patients with systemic lupus erythematosus. *Am J Obstet Gynecol* 160(5): 1227-9, 1989.
5. Schrader, S., Platek, S., Zaneveld, L., Perez-Pelaez, M., Jeyendran, R.: Sperm Viability: A comparison of analytical methods. *Androl* 18(5): 530-8, 1986.
6. Witkin, S., Chaudhry, A.: Relationship between circulating anti-sperm antibodies in women and autoantibodies on the ejaculated sperm of their partners. *Am J Obstet Gynecol* 161:900-3, 1989.
7. Witkin, S.: Failure of sperm-induced immunosuppression: Association with anti-sperm antibodies in women. *Am J Obstet Gynecol* 160: 1166-8, 1989.

**ANTI-TESTICULAR ANTIBODY
TEST SYSTEM**

FOR IN - VITRO DIAGNOSTIC USE

CATALOG NO.:

6748L 48 Test
6796L 96 Test

 SCIMEDX CORPORATION
100 Ford Road
Denville, NJ 07834 USA
Tel: 800.221.5598 Tel: 973.625.8822
Fax: 973.625.8796
www.scimedx.com

EC	REP	Emergo Europe Molenstraat 15 2513 BH The Hague The Netherlands
----	-----	---



SCIMEDX
CORPORATION



*Your Preferred
Autoimmune and
Infectious Disease
Detection Company*

100 FORD ROAD
DENVER, NEW JERSEY 07834 U.S.A.
(973) 625-8822 ♦ FAX: (973) 625-8796

CUSTOMER INQUIRIES:
(800) 221-5598

Printed in U.S.A. Rev. E 10/6/11

6748L.E